neoprair

Protecciones de neopreno transpirable
Breathable neoprene supports
Attelles de Support en neoprene aéré
Protezioni in neoprene traspirante
Proteções de neoprene respirável

NPOS173 · NPOS164 · NPOS167
NPOS165 · NPOS172 · NPOS154 · NP606

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení
Anleitung · Tilpasning instruktion · Οδηγίες εφαρμογής · Fitting instructions · Sovitusohjeet
Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások
Istruzioni per l'adattamento · Montage-instructies · Tilpassingsinstruksjon · Instrukcja obsługi
Instruções de adaptação · Instrucțiuni de reglare · Инструкции по адаптации
Návod na použitie · Utprovningsinstruktion

 PRIM

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Description and intended use:

NPOS172 NEOPRAIR Epicondylitis strap: Support made of three-layer breathable, elastic and padded breathable material with a silicone condylar pad that puts pressure on the tendon and helps eliminate pain.

NPOS173 NEOPRAIR Elbow support: Made of three-layer breathable, elastic and padded material. Applies adjustable compression through Velcro fasteners.

NPOS164 NEOPRAIR Bandage-style wrist support: Made of three-layer breathable, elastic and padded material. Applies compression to the wrist by means of an adjustable strap with Velcro fastening.

NPOS167 NEOPRAIR Wraparound wrist support: Made of three-layer breathable, elastic and padded material. Compresses the wrist using Velcro on the inside to adjust and on the outside to regulate the compression level.

NPOS165 NEOPRAIR Wraparound wrist and thumb support: Made of three-layer breathable, elastic and padded material with two Velcro fasteners to adjust the pressure to the thumb and wrist.

NPOS154 NEOPRAIR Support band: Made of three-layer breathable, elastic and padded material that provides warmth, compression and support to the lower back.

NP606 NEOPRAIR Lumbosacral back support: Made of three-layer breathable, elastic and padded material that provides warmth, compression and support for the lumbosacral region of the spine thanks to two paravertebral stays.

Indications:

NPOS172 NEOPRAIR Epicondylitis strap / NPOS173 NEOPRAIR Elbow support: Lateral epicondylitis (tennis elbow), medial epicondylitis (golfer's elbow).

NPOS164 NEOPRAIR Bandage-style wrist support: Osteoarthritis, arthritis, tendinitis, bursitis, sprains, minor joint injuries.

NPOS167 NEOPRAIR Wraparound wrist support: Sprains, tendinitis and physical activity that puts a strain on the wrist joint. Osteoarthritis, arthritis, bursitis and post injury.

NPOS165 NEOPRAIR Wraparound wrist and thumb support: Rhizarthrosis, arthritis, tendinitis, bursitis, sprains and minor injuries.

NPOS154 NEOPRAIR Support band: Mild lower back pain and when slight compression is required.

NP606 NEOPRAIR Lumbosacral back support: Lower back pain and minor trauma. Sports practice.

Contraindications:

- Do not use on open wounds with swelling.
- Do not use if the area where the orthosis is placed is infected.
- Do not use in the case of skin allergies due to contact with biocompatible materials.
- Do not use if the area starts to redden and/or becomes excessively hot.

Warnings:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the supplier you purchased it from before use.
- For the orthosis to be effective, patients must use the appropriate size. For product NP606, if the patient is close to the maximum range of a size and has a protruding abdomen, choose the next size up.
- Always follow these general instructions for use and any particular guidelines indicated by your doctor.
- Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring.
- If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately. Discontinue use until assessed by a specialist.
- Your doctor should consider the use of topical creams in conjunction with the orthosis.
- The skin should not be damaged or hypersensitive in the areas where pressure is applied.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting and adjustment of these accessories must be carried out by qualified personnel (doctor, orthopaedic technician etc.).
- For the orthosis to function correctly, its components must be regularly checked to ensure they are in good condition. The medical personnel responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.

- Care should be taken to clean the Velcro quick-release fasteners to maintain their functionality, removing any materials that may have adhered to them.
- Do not use in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain adequate personal hygiene.
- Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, make sure you cover it or keep it away from the water. Do not let children play with this product.

Fitting instructions:

When using, follow the advice of the professional who recommended and/or dispensed the product. To gain the highest degree of therapeutic effectiveness for the different conditions and to prolong the life of the product, it is essential to select the right size for each patient or user. As excessive compression can lead to intolerance, we recommend adjusting the compression to the optimum level. Fit the product to the joint in question, making sure that it is properly aligned to enable correct movement of the muscles. Adjust the product to achieve a snug fit with no discomfort.

NPOS172 NEOPRAIR Epicondylitis strap: Position on the forearm about 3 cm below the elbow, allowing full 90° flexion.

NPOS173 NEOPRAIR Elbow support: Insert the arm into the orthosis and position it so that the fastener with the logo is on the upper arm. Adjust the straps to obtain the desired pressure, as shown in Fig. 1.

NPOS164 NEOPRAIR Bandage-style wrist support: Fit around the wrist and adjust the strap to achieve the required compression.

NPOS165 NEOPRAIR Wraparound wrist and thumb support: Fit the narrower part around the thumb and the wider part around the wrist as shown in Fig. 2, depending on the hand.

NPOS167 NEOPRAIR Wraparound wrist support: Insert your thumb through the hole and fasten, then adjust the compression with the fastener that wraps around the wrist.

NPOS154 NEOPRAIR Support band: Position the support at a height of approximately 5 to 7 cm above the beginning of the gluteal fold, centring the coloured vertical lines on each side of the spine. Fasten and adjust the compression level with the Velcro.

NP606 NEOPRAIR Lumbosacral back support: Position the support at a height of approximately 5 to 7 cm above the beginning of the gluteal fold, centring the stays on each side of the spine. Fasten and adjust the compression level with the Velcro.

Product maintenance - washing instructions:

Before washing the support, fasten all of the Velcros and detach the removable components (silicone pads, splints, palmar stays, etc.). Hand wash. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose the orthosis to direct heat sources such as stoves, heaters, radiators and direct sunlight.

Note: Any serious incidents involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.