# Primospine<sup>®</sup>

## GOLD

Fajas semirrígidas  
مشدات شبه صلبة  
Stabilizační bederní pásy  
Halbstarre Stützbandagen  
Semirigide korsetter  
Ζώνες ημίσκληρες  
Pooljäigad tugivööd  
Semi-Rigid back support belts  
Puolijäkät tukiliivit  
Ceintures semi-rigides  
Polukruti steznici  
Félmerev deréktámasztó övek  
Fasce semirigide  
Semi-stevige korsetten  
Halvstive støtteleter  
Półsztywne pasy ortopedyczne  
Faixas semirrígidas  
Bråuri semirigide  
Полужесткие бандажы  
Polopevné pásy  
Medelstyva korsetter  
Poltogi pasovi

PRS640G · PRS641G · PRS642G  
PRS643G · PRS646G · PRS647G  
232BA · 233BA · 234BA

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení · Anleitung ·  
Tilpasning instruktion · Οδηγίες εφαρμογής · Kohandamisjuhised · Fitting instructions ·  
Sovitusohjeet · Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások ·  
Istruzioni per l'adattamento · Montage-instructies · Tilpassingsinstruksjon · Instrukcja obsługi ·  
Instruções de adaptação · Instruțiuni de reglare · Инструкции по адаптации ·  
Návod na použitie · Utprovningsinstruktion · Navodila za prilagoditev

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## ENG

### Description and intended use:

Semi-rigid back support belts made from non-elastic breathable, three-layer textile material that adapts perfectly to the patient's shape. Equipped with steel paravertebral stays and tensioners that allow the level of compression and stabilisation to be adjusted.

### Indications:

**Models PRS640G\_PRS641G\_PRS642G\_PRS643G\_232BA\_233BA\_234BA:**

When extra stabilisation of the lumbar zone is required. Lower back pain, lumbosciatica, spondylarthrosis, lumbar discopathy, herniated disc, osteoporosis, muscle weakness, disc protrusion. Model PRS641G / 234BA is specially designed for patients with a pendulous abdomen.

**Models PRS646G\_PRS647G:**

Provide partial thoracic containment. Kyphotic curve correction, shoulder antepulsion, back pain, degenerative disc processes, wedged vertebrae, spondylolisthesis, spondylitis, postoperative recovery, osteoporosis. Model PRS647 is specially designed for patients with a pendulous abdomen.

### Warnings:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop that sold you the product.
- For the orthosis to be effective, patients must use the appropriate size. If the patient is close to the maximum range of a size and has a protruding abdomen (but not a pendulous abdomen), choose the next size up.
- Always follow these general instructions of use and any particular guidelines indicated by your general practitioner.
- Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring.
- If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately. Discontinue use until assessed by a specialist.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function correctly, its components must be regularly checked to ensure they are in good condition. The medical staff responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.
- To ensure they function correctly, clean the hook-and-loop fastenings regularly, removing any material that may have stuck to them.
- Some of the materials that make up the product are not flame retardant. Therefore, its use is not recommended in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain proper hygiene.
- This product is not designed to be worn in the bath or shower. Ask your doctor if you can remove it, and if not, keep it away from water.
- Do not let children play with this device.

**Contraindications:** None specified.

### Instructions for fitting:

The following aspects should be taken into account when fitting the product:

**Models PRS640G\_PRS642G\_PRS643G\_PRS646G\_232BA\_233BA:**

1. Wrap the orthosis around the body so that the steel stays are centred on the back, taking the spine as the midpoint. The lower part of the belt should be level with the pelvis. The tension straps must remain loose while fitting the main part of the belt.
2. Insert your right fingers into the handle on the right side of the belt and, with your left fingers

## ENG

in the handle on the left side, press the left-hand front panel down over your belly and pull the right-hand front panel over it until the hook-and-loop fastener on one end meets the other and can be secured. Fig. 1 and Fig. 2

3. Pull both ends of the tensioning straps forward to increase the amount of compression. Fig. 3
4. Then, pull and stick the ends of the tensioning straps to the front closure panel of the belt. Fig. 4
5. For the PRS646G dorsolumbar model, adjust the tension of the straps to achieve good shoulder retropulsion and fasten the ends of the straps to the belt's front closure panel. Fig. 5

### Models designed for pendulous abdomen PRS641G\_PRS647G\_234BA:

1. Wrap the orthosis around the body so that the steel stays are centred on the back, taking the spine as the midpoint. The lower part of the belt should be level with the pelvis. The tension straps must remain loose while fitting the main part of the belt.
2. Adjust the belt correctly and close the hook-and-loop fastener on the front left side. As this model is designed for a pendulous abdomen, the hook-and-loop closure should be on the patient's left side, with the abdomen supported by the specially designed cavity, as shown in Fig. 6.
3. Pull the ends of the tensioning straps forward to increase the amount of compression. Fig. 7
4. Then, pull and stick the ends of the tensioning straps to the front closure panel of the belt. Fig. 8
5. For the PRS647G pendulous abdomen dorsolumbar model, adjust the tension of the straps to achieve good shoulder retropulsion and fasten the ends of the straps to the belt's front closure panel. Fig. 9

### Product maintenance - washing instructions:

Close all the hook-and-loop fasteners before washing. Hand wash with warm water and mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

**Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.**

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PRIM, S.A.